

## INSTALL

1. Place collar on a flat surface. Separate left and right collars, rear straps remaining attached.
2. Place stretcher between left and right collar; zipper at head end of the basket with logo visible on patient's right.
3. Clip the five short restraint straps located along right collar to basket. Repeat process with left collar. Tighten restraint straps, as required.
4. Flip stretcher on its front and adjust the rear straps, as required.

## STORE

1. Unclip rear straps on underside of basket.
2. Turn basket back over and flip collars into basket



**Manufactured by Manutex**  
5200 Dixie Rd., Unit 11  
Mississauga, ON L4W 1E4  
(905) 238 - 2766  
**For Traverse Rescue LLC.**